



Instructions Stool Assembly

1.  Model No.:MST01  Model No.:MST03

Place the button end of the adjustable cylinder into the opening in the seat tube. Use minimal force to press the cylinder into the seat tube.

2.  Model No.:MST01  Model No.:MST03

Press one castor into each of the ends of the five-point base.

3.  Model No.:MST01  Model NO.:MST03

Set the chrome-metal end of the adjustable cylinder into the seat tube in the five-point base.



Model NO.:MST01



Model NO.:MST03

Use minimal force to press the adjustable height cylinder into the base.
Carefully sit or put pressure on the seat until the cylinder locks into place.



Model No.:MST01



Model No.:MST03

Using the lever under the seat, you can adjust the stools height to a spot that best suits you.

Caution: Max weight rating: 250lbs
Fragile Handle with care.