

## Nasal Strips

The main effect is to improve sleep disorders caused by nasal congestion and temporarily reduce or relieve snoring.

### **Description :**

Structure:

1) Two flexible "spring-like" pieces gently lift upwards the congested nasal passages so you can breathe through your nose, close your mouth and stop snoring, allowing you and your partner to get a good night's rest

2) Special adhesive holds the strip comfortably in place

3) Improves airflow by 31%

Material: PE, nonwoven cloth and film

Capabilities:

1) Peel off intensity: >1.0 N/cm

2) Air permeation rate: >200g/m<sup>2</sup> 24H

3) Normally no rash or skin irritation

Size is important, get the right size for maximum advantage (55mm x 18mm, 58mm x 16mm, 66mm x 19mm, 73mm x 19mm).

### **Function :**

Expand nasal passages;

Used for relief of nasal congestion caused by cold, allergy and rhinitis;

Reduce snore and improve breathing.

### **Cautions:**

1) To get maximum results, please place strip correctly

2) For single use only, do not use for more than 12 hours

3) Not for children under 5 years old

4) Do not use over sores, sunburned or irritated skin

5) If rash or skin irritation occurs, discontinue use